



# HawkGrips Cupping Set Cleaning Instructions:

[According to the CDC](#), a HawkGrips Cupping Set is considered a non-critical medical device and requires low-level or intermediate-level disinfection.

**Recommendation:** HawkGrips Cups can be cleaned with neutral, unscented soap and water or disinfectant wipes between each use to remove dirt, oil, and debris from the surface.

***For high-level disinfection:***

Use a non-abrasive disinfectant (i.e. Sporox), and follow these steps:

1. Clean the cups thoroughly with a neutral, unscented soap and water.
2. Immerse the cups in the disinfectant solution, ensuring that they are completely submerged. The recommended contact time is 20-45 minutes.
3. After the specified contact time, remove the cups from the solution and rinse them thoroughly with sterile water to remove any residual disinfectant.
4. Allow the cups to air dry completely before storing or using them.

**If you have any further questions, please contact:**

Email: [info@hawkgrips.com](mailto:info@hawkgrips.com)

Phone: 484-351-8050

